NEDERLANDS

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User's manual

SanoSkin®-OXY is a medical device

Presentation: 30 gram tube + insert in carton box

Product description

SanoSkin®-OXY is a fatty ointment that consists of ozonated vegetable oil (100% olive oil). When the ointment comes into contact with wound fluid/water, ozonides gets released. Those ozonides create an acid and oxidative environment that is unfavorable for micro-organisms like bacteria, yeast and fungi.

Full quantitative and qualitative composition

SanoSkin®-OXY consists of 100% ozonated olive oil and the composition below may vary slightly because it is a natural product.

Nr.	Component	%	margin %
1	Palmitic acid	11.8	+/- 0.05
2	Palmitoleic acid	0.75	+/- 0.05
3	Stearic acid	3.6	+/-0.05
4	Oleic acid	58	+/- 2.0
5	Linoleic acid	8	+/- 1.0
6	Nonanal	4	+/- 1.5
7	Pelargonic acid	3.5	+/- 1.0
8	Azelaaldehydic Acid	4	+/- 1.0
9	Capro aldehyde	2.09	+/- 1.0
10	Azelaic Acid	3	+/- 1.0
11	Non-3-enoic acid	0.3	+/- 0.1
12	Caproic acid	0.05	+/- 0.02
13	Undecylic acid	0.1	+/- 0.05

Indications and Intended use

SanoSkin®-OXY is indicated in the treatment of chronic wounds (Pressure ulcers, various types of ulcers), infected wounds, acute wounds, oncological wounds and minor burns (1st and 2nd degree).

The ointment creates a moist oxidative wound climate, which stimulates the process of autolytic debridement.

SanoSkin®-OXY can be used both by healthcare professionals and laymen.

Contra-indications

Patients with a greatly increased risk of infections such as very extensive burns.

Patients with known hypersensitivity to SanoSkin®-OXY or its components.

Do not use on eyes and/or mucous membranes.

Side Effects

SanoSkin®-OXY can cause in some cases a short stinging sensation.

If a serious incident has occurred with regard to the medical device (SanoSkin®-OXY), the patient/user should report this to the manufacturer and the competent authority of the Member State where the user and/or patient.

Precautions

- Do not use if the tube has been damaged.

- Do not use the product continuously for more than 30 days.
- The presence of bacteria in wounds is a normal phenomenon and is no contra indication for use.
- If an infection is suspected or is developing during the use then a correct antibiotic therapy should be started.
- Depending on the underlying pathology, the wound care treatment should always be accompanied by the correct medical treatment and supervision.
- SanoSkin®-OXY creates a moist wound climate that stimulates the debridement of necrotic tissue. Due to this, it is normal that the wound surface becomes bigger.
- When the condition of the wound deteriorates, treatment with SanoSkin®-OXY should be stopped.
- We recommend that a tube is used on one patient to avoid cross-contamination (= single patient device).

Pregnancy, breastfeeding and use in Paediatric patients (0-15 years)

There is no information available about the use of SanoSkin Oxy during pregnancy and breastfeeding. Are you pregnant or breastfeeding? Please contact your doctor or pharmacist. No contraindications are known to date for the use of SanoSkin®-OXY in Paediatric patients. Use SanoSkin®-OXY always exactly as your doctor or pharmacist has told you, or how it is explained in these insert.

Application

1. Clean the wound.
2. Open the tube aseptically, avoiding contact with other objects.
3. Remove the aluminum seal and squeeze the tube to obtain ointment.
4. Apply the ointment immediately on/in the wound without the ointment rising above the wound edges.
5. Cover the ointment/wound with a moisture-retaining dressing.

Remove

SanoSkin®-OXY should be changed when leakage of the secondary dressing occurs or when the secondary dressing needs to be removed. The ointment can be cleansed out of the wound with a sterile saline solution. Dressing change is depending on the wound fluid production of the wound, but is usual between 2 -7 days.

Waste processing/disposal

Cardboard box, paper leaflet for paper waste and the plastic tube for recyclable waste.

Storage precautions

Store between 5°C - 25°C.

After opening the package, the product can be used for 28 days.

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© SanoMed Manufacturing BV 2022 Barring Spelling errors, SanoMed Manufacturing BV shall at all times retain the right to change its products and assortment.